

FRONT

JORDAN LOUNGE SLEEPERS



krug

HEALTHCARE SOLUTIONS


BACK

LOUNGE POSITION

OPENING




LOCK CASTERS IN FORWARD POSITION,
PULL ON HANDLE AWAY FROM CHAIR.

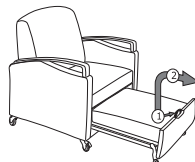


PULL THE STRAP UPWARDS
AND AWAY FROM CHAIR.

CHAISE POSITION

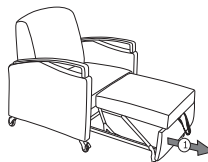


PULL FIRMLY ON HANDLE AWAY FROM
CHAIR TO EXTEND TO SLEEP POSITION.

SLEEP POSITION

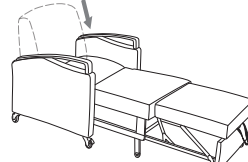


FOLD BACK DOWN INTO
SLEEP POSITION


CLOSING

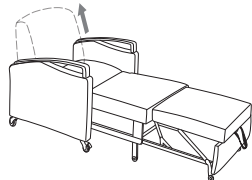

ROTATE BACK UP INTO
SITTING POSITION.

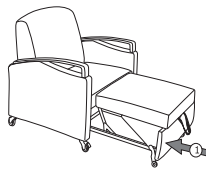

PUSH FRONT PANEL INTO
CHAISE POSITION.

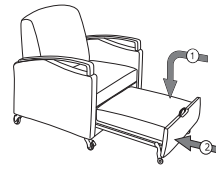

LOWER OTTOMAN TO TUCK AWAY POSITION
PUSH FRONT PANEL INTO CHAIR POSITION.

krug

solutions@krug.ca | www.krug.ca | 1.888.578.KRUG