

JORDAN ACTIVE PATIENT RECLINER



krug

HEALTHCARE SOLUTIONS

OPTIONAL FEATURES



OTTOMAN PROTECTIVE COVER
This optional clear protective cover is sewn into (and over top of) the lower portion of the upholstered ottoman, providing a layer of protection from marring and damage caused by footwear, cleaning and medical equipment, etc. It has an anti-fog additive to maintain clarity, and is resistant to cleaning solutions.



PATIENT TRANSFER ARM
The Patient Transfer Arm is designed to fold down where the arm is level with the chair seat. This design allows the caregiver to easily transport patients from the chair to bed or wheelchair. The arm is folded down by accessing the adjustment lever at the side of the chair. The Patient Transfer Arm can only be specified in the right handed position based on facing the unit.



THERMAL COMFORT
Providing warmth to patients undergoing chemotherapy and other treatments can be vitally important. Thermal Comfort is operated with a hand-held controller located on the inside arm of the recliner. It provides two heat zones: one in the seat and one in the back. Back and seat zones can be controlled independently, at a high or low heat setting. Thermal Comfort features a safety time-out that automatically shuts off heat after 30 minutes of use.



FLIP UP TABLET
The tablet's size is 18" x 14" x 3/4", located on the right side (facing) – opposite the ottoman activation lever on the left side. It is available in Beech and Maple veneer, as well as plastic laminate with a polymer edge. It folds flat (protruding 3" from the side of the recliner when flat).



PUSH BAR
The optional push bar is ergonomically designed to assist care-giving staff in the movement and operation of the recliner, and reduce back strain on the caregiver.



REMOVABLE & REPLACEABLE UPHOLSTERY
Jordan Active Patient Recliners are available with Removable Seat and Back covers. Replacement covers are also available and can be easily field installed.



All products are air emissions certified and are available as FSC® certified

krug

solutions@krug.ca | www.krug.ca | 1.888.578.KRUG



Four positions: back upright with ottoman closed, or extended; back reclined (infinite position range) with ottoman closed, or extended.




The Jordan Active Patient Recliner is lighter-scaled, lower weight and dimensionally smaller than most recliners, but maintains comfortable proportions for extended sitting, resting and treatment. In the maximum reclined position with the ottoman extended, the overall floor space used is only 57" in length, and it has a depth of only 27" in the upright position. Combined with an outside width of only 28" (in the smaller of the two standard widths), and lesser weight, the Active Patient is a compact recliner with the proportions of a patient chair, while providing excellent comfort and a full range of features and options. Its smaller size and weight make it less burdensome to move and live with: at just over half the weight of our Jordan 3-Position full-scale recliner, the Active Patient is a recliner that is easier on caregivers, patients, and cleaning staff. It's the perfect solution for patient rooms and treatment areas where available space and freedom of movement are important.

STANDARD FEATURES



INFINITE POSITION BACK

The back can be reclined and locked in any position within a range of 25 degrees from the upright position, with or without opening of the ottoman. The back recline features a gas-assist mechanism that eases the movement of the back both as it's reclined and as it is returned forward.

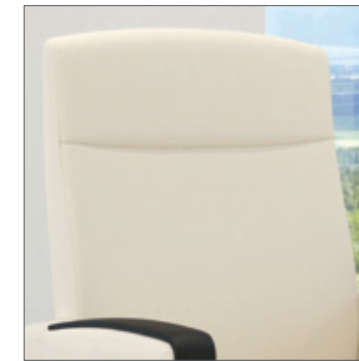

21" AND 24" WIDTHS

The 21" model is well-suited to compact space requirements, with an outside width of 28", and an interior width of 20.5". The 24" model is more generous, with 3" of additional width in both the interior and exterior dimensions.



EXTENDED SITTING COMFORT

While the Active Patient Recliner is a compact recliner solution, no sacrifice in comfort is made: arms are generously proportioned and the seat and back feature dual-density contoured foam that provides the ideal combination of softness and firmness for comfort in all durations of sitting, resting and treatment.



INDEPENDENT BACK & OTTOMAN MECHANISMS

Two independent mechanisms provide a greater range of function and positions for the Active Patient Recliner: the back can be upright or reclined, independent of the ottoman which can be closed or extended separately from the back.



BACK ACTIVATION LEVER

The Active Patient Recliner comes with the back adjustment lever which creates an infinite number of back adjustments within the chair. The back adjustment lever comes in a black plastic finish on the caregiver side.




OTTOMAN ACTIVATION

The adjustment lever is situated to allow both the user and the caregiver to safely operate the ottoman function of the mechanism. The ottoman is generously padded for comfort, and can be given added protection with the optional Ottoman Protective Cover.




URETHANE ARM CAP

Arms have urethane arm caps. The textured urethane provides enhanced durability and protection for the arms and is available in three color options: Black, Grey and Taupe. The urethane arm cap is field replaceable if damaged or degraded.

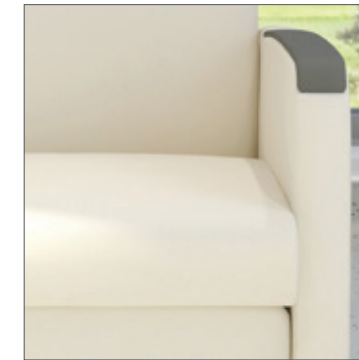

CLEANING & INFECTION CONTROL

Specifically designed to ease the cleaning and disinfection of the recliner and the floor underneath. Upholstery covers can be made removable, and individual components are automatically removable, to enable thorough cleaning and disinfection.

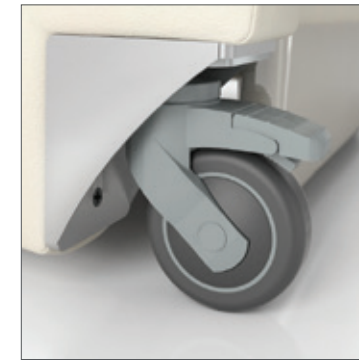

CASTERS

Four heavy-duty 3" diameter soft wheel casters are standard on the recliner. The front casters are inline and locking; the rear casters are swivel and not visible. This combination provides a noticeable and user-friendly ease of steering when the recliner is being moved. Casters are field-replaceable if damaged.

URETHANE ARM CAP COLOR OPTIONS



BLACK

GREY

TAUPE