

Operating Instructions

1

Recline Controls

Located on the side of the arm, the Ascend hand wand is used to control both the back recline and the footrest. The top two buttons control the back, the middle two buttons control the ottoman and the lower two buttons control the back and ottoman simultaneously.

2

Lift Controls

Located on the side of the arm, the Ascend lift control is used to move the unit up and down. Simply press on the up or down button until you reach your desired height. **Do not operate the lift while the unit is in Trendelenburg mode. If applicable stow the swing tablet before use.**

3

Patient Transfer Arm (Optional Feature)

To activate this feature pull down on the lever located at the front of the arm. This will un-lock the arm and allow you to swing the upper portion of the arm down and out of the way. Once you are done using it lift the arm back into the upright position and pull up on the locking lever. **When the arm has been re-locked gently wiggle it back and forth to ensure it is secure.** If the transfer arm is excessively loose or is very hard to lock adjust the built in adjustment screw located inside the upper portion of the arm. Loosening the screw will make the arm easier to open while tightening it will make it lock more firmly.




1



2



3

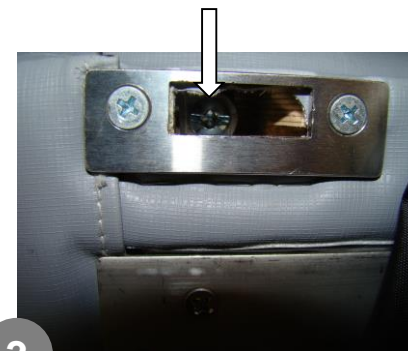

3

Operating Instructions**4****Trendelenburg Lever (Optional Feature)**

Ensure the unit is in its lowest position before operating the Trendelenburg feature.

To convert the unit into Trendelenburg mode press down on the red foot activated pedal below the arms. Once the pedal is pressed gently push down on the back. Releasing the foot pedal will engage the lock and ensure that the chair stays in place. To get the chair back into the standard position press down on the foot pedal again and gently press down on the seat until it is horizontal.

5**Stirrups (Optional Feature)**

The stirrup bars are located just below the seat. Using one finger, pull them out to the desired length. Remove the stirrups from the holder on the side of the arm and slide the stirrup onto the stirrup bar.

6**Paper Roll Holder (Optional Feature)**

This feature accepts an 18" wide roll of exam paper. To load/unload rolls untighten the black knobs found on either end of the bar. Remove the bar, and slide it thru the roll of paper. Re-install the bar ensuring that the thumbscrews are tight.



Operating Instructions**7****PowerDoc (Optional Feature)**

Press down on the inner front edge to pop the unit out. Once you are done using it press back down until an audible click is heard.

8**Swing Away Tablet (Optional Feature)**

To use, rotate the tablet up to a minimum of 90° from it's resting position and lower it in front of the patient so it is sitting parallel to the floor. To stow the tablet flip it up so the tablet surface is parallel to the side of the arm and rotate it back to it's resting position. **Care should be taken when stowing the tablet. Do not allow it to fall freely as this may cause damage to both the unit and the tablet.**

9**Flip Tablet (Optional Feature)**

To raise the tablet, pull up on it until it is horizontal and "clicks" into place. To lower, depress both safety latches located on the tablet bracket and slowly lower the tablet into it's resting position.

**7****8****9**